



Assist-A-Grip

JFC Manufacturing Inc.


Instructions for using strap

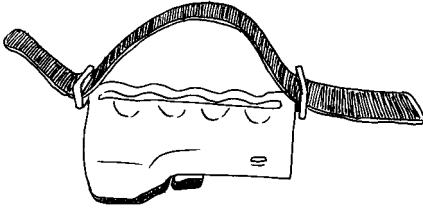


A. Open strap — make sure to bow upward in the middle section.

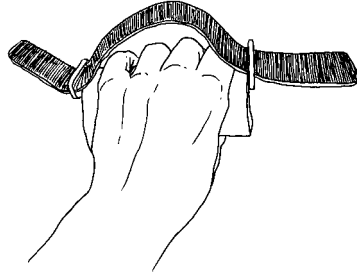

B. Slide hand between Assist-A-Grip and the Velcro strap.

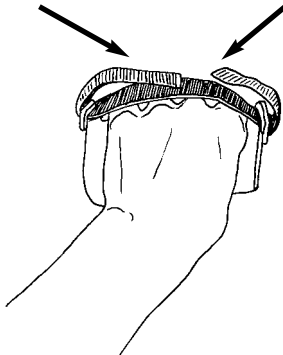

C. Start to tighten by bringing the Velcro ends over your knuckles.

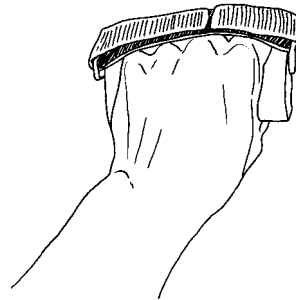

D. Tighten strap.

JFC Manufacturing Inc.

P.O. Box 702
Lititz, PA 17543

Toll Free: (866) 458-9885

Fax: (717) 626-4513

Website: www.assistagrip.com

E-mail: sales@jfcmanufacturinginc.com